Galaxy Mercury

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User's manual

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- i Basic information** 15
 - General notes 15
 - Safety notes 15
- Specification** 15
 - Composition and construction of the device 15
 - Technical parameters 17
 - Markings 17
 - Marking on the packaging 17
 - Marking on the device 17
 - Marking inside the device 17
- Operation** 17
 - Application 17
 - User's obligations 17
 - Conditions for use of the device 18
 - Language and program selection 18
 - Touch panel 18
- Treatments** 18
 - Microdermabrasion** 18
 - Principle of the operation 18
 - Indications for treatment 18
 - Contraindications for treatment 18
 - Technical parameters 18
 - Device panel 18
 - Settings of treatment parameters 19
 - Maintenance and disinfection 19
 - Ultra Sonic** 19
 - Principle of the operation 19
 - Indications for treatment 19
 - Contraindications for treatment 19
 - Technical parameters 19
 - Device panel 19
 - Cavitation Peeling setting 19
 - Ultra Sonic setting 19
 - Settings of treatment parameters 20
 - Cavitation Peeling 20
 - Ultra Sonic HF 20
 - Maintenance and disinfection 20
 - Radio Frequency** 20
 - Principle of the operation 20
 - Indications for treatment 20
 - Contraindications for treatment 20
 - Technical parameters 20
 - Device panel 21
 - Settings of treatment parameters 21
 - Maintenance and disinfection 21
 - Mesotherapy** 21
 - Principle of the operation 21
 - Indications for treatment 21
 - Contraindications for treatment 21
 - Technical parameters 21
 - Device panel 21
 - Settings of treatment parameters 22
 - Maintenance and disinfection 22
- Tips for safe device operation and correct exploitation** 22
 - Transport 22
 - Problems with device operation 22
 - Inspections and checks of technical condition 24
 - Device operating life and disposal 24
- Warranty conditions** 24

i BASIC INFORMATION

Dear Client!

Congratulations on making the right choice. We wish you plenty of satisfaction with the operation of our device. Before you start, please carefully read this operating manual. The manual contains important information as well as manufacturer's instructions regarding correct installation, exploitation and maintenance of the product.

GENERAL NOTES SAFETY NOTES

- Rubica Galaxy Mercury is a cosmetic device.
- The device is not a medical equipment. It is not designed in any way for diagnosing, treatment or prevention of diseases. If the Client is unsure if she/he can undergo the treatment, she/he should seek medical advice from a general practitioner.
- Operation, handling and maintenance of the product not in accordance with this operating manual is prohibited and can result in damage, for which the manufacturer is not liable.
- If device operation and parameters are inconsistent with the description contained in this manual, do not use the product, and report it immediately to the manufacturer or supplier.
- Every repair of the device must be carried out by factory service and properly recorded. Failure to comply with this requirement will void the warranty on the product.
- Do not keep the head of the device too long in one area as this may result in a nociceptive (painful) sensation in the Client.
- Do not remove the head from the skin during treatment.
- The skin should be carefully monitored during treatment.
- Treatment should never be accompanied by pain.
- The previously used treatment heads should never be used in another Client without prior disinfection.
- Treatment should be immediately discontinued if the client feels discomfort.
- Prevent heads from falling or being struck.
- Rubica Galaxy Mercury is designed for indoor use only.
- Never switch on the device with disconnected heads. This may result in damage not covered by the warranty.
- Do not open the top cover otherwise the warranty may be null and void.
- Do not modify or repair the device by yourself.
- Do not use volatile substances (e.g. solvents, gasoline) to clean the device.
- Do not use alcohol-based preparations to clean or disinfect the touch screen.


„Caution, consult accompanying documents”

This sign has been put on the casing of the device as its operation contrary to this operating manual may pose a threat to the safety of either the person undergoing treatment or the operator of the device.

- Prior to the initial use, carefully read this operating manual.
- Do not kink or pull the cable that feeds electricity to the central control unit.
- The device must be connected to a standard earthed 230 V/50 Hz power grid.
- Do not approach the device to water. Direct contact with water is forbidden, as it may cause electric shock.
- Do not forcefully pull out the mains plug.
- During treatment, it is forbidden to cut off the power supply with the main switch situated at the back of the device. This may result in the occurrence of maximum available impulses on the head and unpleasant sensations to the Client.
- After treatment and prior to cleaning, the device must be switched off and the mains plug must be pulled out of the power socket.
- Keep out of children's reach.
- Protect from moisture, water and dust.
- Keep the device away from convectors and other sources of heat.
- Do not place the device in a tightly sealed container, do not cover ventilation holes.

Specification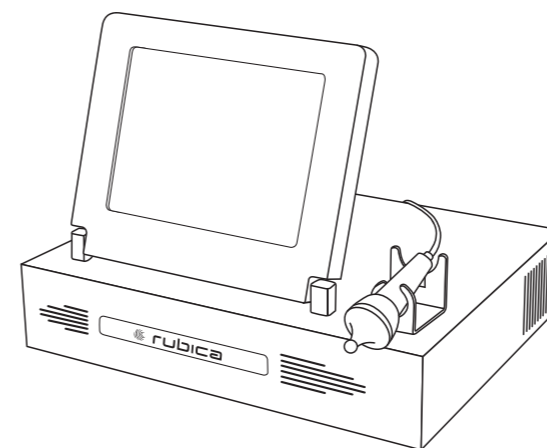

COMPOSITION AND CONSTRUCTION OF THE DEVICE


Components

1. Main unit – 1 pc.
2. Operating heads for microdermabrasion – 10 pcs.


3. Operating heads for Ultra Cavi Sonic and Mesotherapy


a. Small – Eye (Sonic+Meso)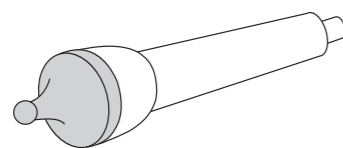


b. Medium – Face/Body (Sonic+Meso)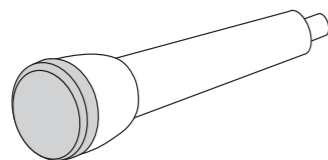

c. Large (Sonic)

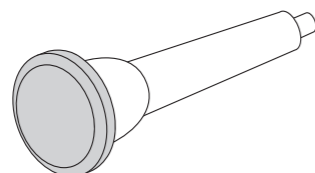
4. Spatula (Sonic)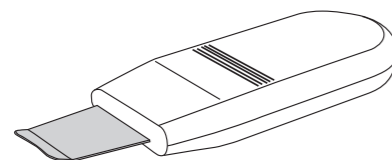


5. Operating heads for Radio Frequency – 3 pcs.


a. Small – Eye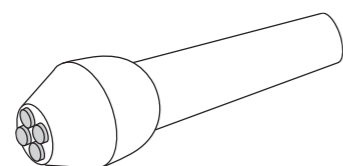


b. Medium – Face/Body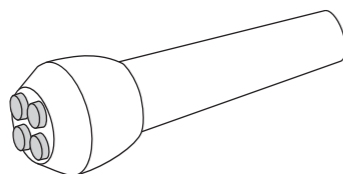

c. Large – Body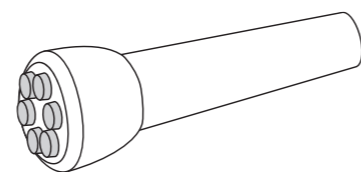


6. Nozzles for operating heads use for microdermabrasion – 3 pcs.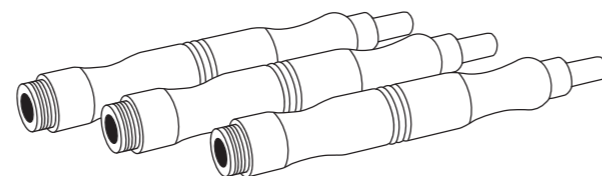

7. Spare filter for microdermabrasion – 2 pcs.

8. Suction tubes – 3 pcs.

9. Passive electrode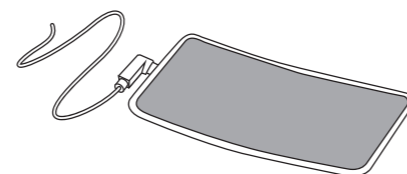

10. Power supply cable – 1 pc.


11. Guarantee card – 1 pc.

12. User's manual – 1 pc.

13. Spare fuses T1A/250 V – 2 pcs.


14. Touch screen

Main unit – front view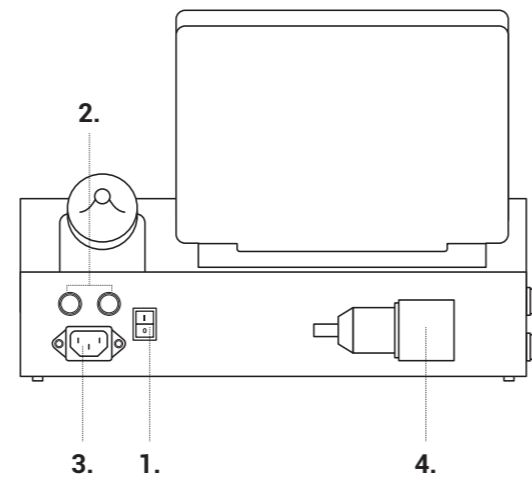

1. Power switch
2. Fuse socket
3. Supply socket for 230 V
4. Replace filter inlet for microdermabrasion

Main unit – side view

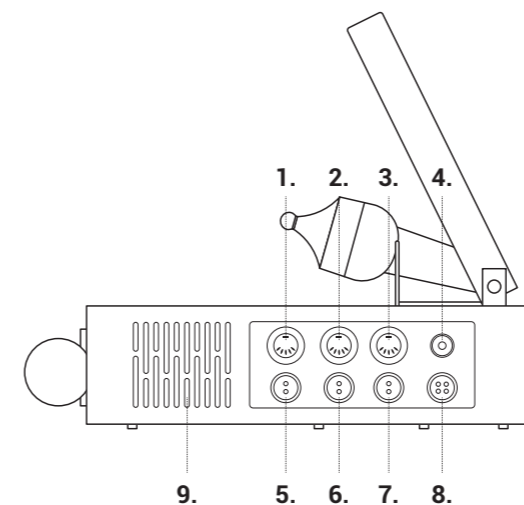1. RF operating head small
2. RF operating head medium
3. RF operating head large
4. Passive electrode
5. Sonic/Mezo conic operating head
6. Sonic/Mezo operating head medium
7. Sonic operating head
8. Sonic spatula
9. Ventilation holes

TECHNICAL PARAMETERS

- **Dimensions:** width 34 cm; depth 30,5 cm; height 11,5 cm – folded screen; 26 cm – open screen
- **Weight (including heads):** 8.4 kg
- **Voltage:** 230 V +/- 10%, 50 Hz +/- 2%
- **Application part:** type B acc to PN-EN 60601-1:1999
- **Protection rating:** I

OPERATION

APPLICATION

- Rubica Galaxy Mercury device is broadly used in beauty salons, SPAs, and health and beauty centres.
- **Only persons with cosmetic education and suitable training in the operation of the Galaxy Mercury device** can connect the device and perform cosmetic treatments.
- Personnel operating the device and performing cosmetic treatments must first read this operating manual and adhere to the instructions, warnings and notes contained therein.

USER'S OBLIGATIONS

- **User** – any person who owns tangible property as owner, lessee, pledgee, renter or with other rights in the object, as well as any person who uses

MARKINGS

Marking on the packaging


upside, keep upright

keep dry

fragile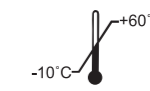

temperature limitations

Marking on the device

CE marking

application part: type B

caution, consult accompanying documents

This symbol on the product means that used electrical and electronic products should not be mixed with general household waste.

date of manufacture

>150 VA

maximum input power

230 V AC
50 Hz

power supply

SN

serial number

Marking inside the device

protective earthing

the product independently or for whom the product is used.

- The user is obliged to ensure that the product has been used only for its intended purpose and that it is operated under the conditions provided for and in accordance with this operating manual.
- The user is obliged to take all necessary measures to ensure safe operation of the product and to prevent any threat to life and health of the user, Clients and third parties.
- Only authorized personnel with adequate training and familiar with this operating manual is permitted to operate the product. The user must ensure that all operators read, understand and adhere to this operating manual.

CONDITIONS FOR USE OF THE DEVICE

The product is designed for indoor operation, in the absence of condensed steam.

- **temperature:** from 10°C to 40°C
- **relative air humidity:** from 30% to 80%
- **atmospheric pressure:** from 700 hPa to 1060 hPa

LANGUAGE AND PROGRAM SELECTION

1. Turn on the device with a switch on the back of the device.
2. Select language on the touch panel.
3. Confirm by pressing **Start/Stop**.
4. Select device type from the main menu.

TREATMENTS

MICRODERMABRASION

PRINCIPLE OF THE OPERATION

Wrinkles, discoloration, scars, acne scarring are symptoms of skin problems and aging. One way to address them is diamond microdermabrasion – a completely non-invasive and efficient form of mechanical peeling. This method is an alternative to chemical peeling and aesthetic medicine treatments. This treatment involves physical removal of the horny layer of the epidermis to the required level. The depth of microdermabrasion depends on several factors, such as: selected negative pressure, treatment head diameter, diamond gradation and length of treatment in one area. Precise negative pressure settings allow for treatment of sensitive and thin skin. Peeling without the need to use chemicals minimises the risk of allergies. Immediately after treatment, skin tone is visibly improved, wrinkles are smoothed, the skin is brightened, scars and discolorations are less visible. The long lasting effect of treatment is significant, as it initiates changes in the skin itself. The stimulation of collagen and elastin synthesis have a positive effect on thickness, firmness and condition of the skin. After peeling treatment, it is advisable to use professional cosmetics selected according to skin type, which will intensify and prolong the effects of treatment. Diamond microdermabrasion is an excellent preparation for the skin for absorption of active ingredients, and therefore it can be performed before all care treatments. It is particularly recommended where the Client is not fit for treatments using electricity or ultrasound. All benefits of diamond microdermabrasion and the fact that it does not require any sacrifices in everyday lifestyle have made this treatment one of the most commonly performed procedures in cosmetic salons chosen by Clients.

INDICATIONS FOR TREATMENT

- + blackheads
- + wrinkles and fissures
- + acne scars
- + skin laxity
- + seborrhea
- + atrophic and hypertrophic scars
- + dilated pores
- + stretchmarks
- + discolorations
- + keratosis pilaris
- + sun damaged skin
- + rough, horny epidermis
- + skin rejuvenation
- + preparation of skin for laser treatments, mesotherapy, surgical cosmetic procedures

TOUCH PANEL

- The new RUBICA touchscreen makes the operation of the device easy and efficient.
- It is based on state-of-the-art technologies.
- It allows precise adjustment of treatment parameters, that's why its operation requires particular caution.
- Avoid pressing hard on the screen. Striking or exerting excessive pressure on the panel may damage the screen.
- Be sure to touch the screen using gentle and short taps with your finger tip.
- To clean and disinfect the screen use the non-alcohol based preparations.


CONTRAINDICATIONS FOR TREATMENT

- x viral and bacterial skin infections
- x purulent acne
- x acne rosacea
- x broken epidermis
- x skin tumours
- x moles, angioma
- x proneness to keloids
- x retinoid acne treatment (up to 6 months after completion of treatment)

TECHNICAL PARAMETERS

- Negative pressure: 16–76 kPa (5–23 inHg)
- Accuracy of negative pressure setting:
 - <56 kPa: +/- 3 kPa
 - > 56 kPa: better than +/- 15% of the setting
- Operating time: 1–40 min

DEVICE PANEL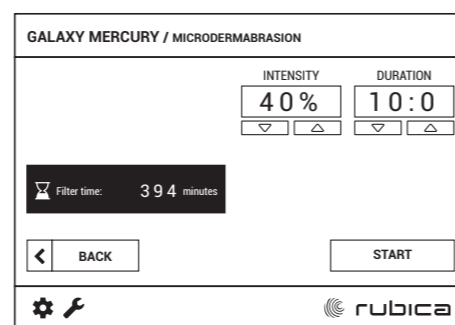

Device panel – **Microdermabrasion** setting

1. Decreasing treatment time (-)
2. Increasing treatment time (+)
3. **Back** – back to the selection menu
4. Decreasing negative pressure (<)
5. Increasing negative pressure (>)
6. **Start/Stop** – power switch/Operating mode selector

SETTINGS OF TREATMENT PARAMETERS

1. Select **Microdermabrasion** on the panel.
2. Adjust **operation time** with buttons (1) and (2). Press the **Start/Stop** button (6). Put your finger on the device head and adjust the negative pressure with buttons (4) and (5).
3. After the above steps are completed, microdermabrasion will start. The device head should be constantly in motion. Do not keep it in one place.
4. End of treatment is signalled by a short acoustic signal.

MAINTENANCE AND DISINFECTION

After treatment, wipe the operating heads dry and disinfect using Hospisept-Tuch. Do not soak. Clean the touch panel with foam designed to cleaning LCD screens. The remaining parts of the device should be cleaned with Anios D.D.S.H. active foam. Never use alcohol-based products for this purpose. Treatment heads should be disinfected after each use.

Stage I – disinfection and washing of treatment heads in the Aniosyme D.D.1 solution. It is recommended to use an ultrasonic cleaner.

This eliminates the need to remove organic residue by brushing while ensuring thorough cleaning of the treatment head.

Stage II – optional:

- High-level disinfection with ready to use Steranios 2% liquid with anti-spore effect. It is possible to check reusable this effect with validated control strips..
- Sterilization in autoclave (this method may cause some dark discoloration on the treatment head or faster crushing of the surface of the diamond head).

Nozzles and tube should be disinfected after each use with Aniosyme D.D.1. and left to dry so the residual liquid can easily escape.

ULTRA SONIC

PRINCIPLE OF THE OPERATION

Ultrasounds are elastic waves with frequencies above the audible range for humans, exceeding 20.000 Hz. This technology was promoted and gained popularity in the 1980s, owing to its effectiveness, wide range of application and ease of use.

Ultrasounds are becoming an increasingly popular for skin cleansing, referred to as cavitation peeling. An ultrasound wave travelling in an aqueous environment, which usually is a delicate tonic, creates tiny bubbles filled with thin air. They expand and suddenly burst under the effects of pressure differences. This results in skin cleansing of residual horny layer of epidermis, toxins and sebaceous gland excretions, thoroughly and completely pain free.

This method does not require heat application and is therefore safe for sensitive skin and telangiectasia. The peel ideally prepares the skin for further care treatments, increasing the absorption of applied cosmetics. A basic effect of ultrasound on tissue is a mechanical influence known as micromassage, caused by pressure fluctuations as the ultrasound wave travels. Low strength and frequency vibrations are emitted from special treatment heads which massage skin covered with an appropriate gel (transmission/contact gel). During treatment the contact gel provides excellent grip for the operation head to the skin, making for smooth movement and undisturbed ultrasound wave travel deep into tissue. Thanks to these vibrations particles in the skin and subcutaneous layers of tissue vibrate and warm up, blood vessels dilate, increasing the supply of blood to cells, and with it, supply of oxygen and other nutrients. The permeability of cell membranes increases and this is exploited in the application of active ingredients contained in cosmetics. These are chosen from a wide range of moisturizing, nourishing, balancing, anti-wrinkle, anti-cellulite and slimming cosmetics, according to the treatment area and skin problem being addressed.

INDICATIONS FOR TREATMENT

- + blackheads
- + contaminated skin
- + wrinkles and furrows

- + grey complexion
- + discolorations
- + skin in need of regeneration
- + oxygen-starved skin
- + swelling around the eyes
- + slimming
- + cellulite removal


CONTRAINDICATIONS FOR TREATMENT

- x pregnancy
- x cancer (up to 5 years after treatment completion)
- x heart disease, hypertension
- x viral and bacterial skin infections, broken epidermis
- x metal implants in treatment area
- x pacemaker, replacement heart valve
- x epilepsy, multiple sclerosis
- x acute sinusitis
- x immunological illness
- x hyperthyroidism
- x fever
- x general exhaustion

TECHNICAL PARAMETERS

- Operation time: 1–30 minutes
- Ultrasound frequency:
 - Cavitation peeling head – 25 KHz
 - Eye, Face, Body heads – 1 MHz
- Power output: 0.2 W/cm²–1.6 W/cm²
- Pulse rate: 1 Hz, 2 Hz, 5 Hz, 10 Hz, 20 Hz

DEVICE PANEL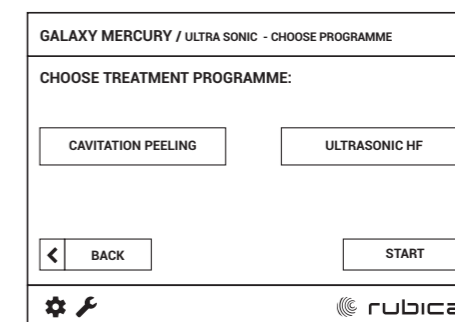


Cavitation Peeling setting

1. Ultrasound intensity setting (-)
2. Ultrasound intensity setting (+)
3. Time adjustment (-)
4. Time adjustment (+)
5. **Back** – back to the selection menu
6. The device power setting (<)
7. The device power setting (>)
8. **Start/Stop** – power switch

Ultra Sonic setting

1. Ultrasound intensity setting (-)
2. Ultrasound intensity setting (+)
3. Time adjustment (-)
4. Time adjustment (+)
5. **Back** – back to the selection menu
6. The device power setting (<)
7. The device power setting (>)
8. **Start/Stop** – power switch
9. Operating head selection

SETTINGS OF TREATMENT PARAMETERS

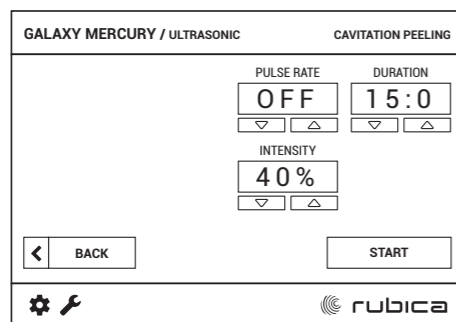CAVITATION PEELING

1. Select on the panel *Ultra Sonic* and next *Cavitation Peeling*.
2. Adjust *power, ultrasound wave cycle* and *set desired time*.
3. Approve the settings by clicking *Start/Stop* (8).
4. Put the spatula into the treatment area, keeping the skin well moistened with tonic.
5. The treatment clock maintains time until the end of the treatment.
6. End of treatment is signalled by a short acoustic signal.

ULTRA SONIC HF

1. Select on the panel *Ultra Sonic HF*.
2. Adjust *ultrasound wave cycle* with (1) and (2) button, *desired time* with (3) and (4) button, with button (9) choose the right *treatment head*.
3. Put the head of the device into the body.
4. The head of the device should be all the time in motion. Do not keep it in one place.
5. Press *Start/Stop* button (8).
6. With (6) and (7) button adjust *the power*, appropriate for the Client.
7. After having finished the above steps, the treatment starts.
8. The treatment clock maintains time until the end of the treatment.
9. End of treatment is signalled by a short acoustic signal.

MAINTENANCE AND DISINFECTION

After treatment, wipe the treatment head dry, dismantle and disinfect with Hospisept-Tuch. Do not immerse!

The touch-panel should be cleaned with foam designed to cleaning LCD screens. The remaining parts of the device should be cleaned with active foam Anios D.D.S.H. Never use alcohol based products for this purpose.

Wipe the cavitation spatula and all operating heads dry and disinfect with Hospisept-Tuch. Do not immerse!

RADIO FREQUENCY

PRINCIPLE OF THE OPERATION

Radio Frequency is a rejuvenating and slimming treatment. It involves the radio waves – (bipolar) alternate high-frequency currents. The method – recently exclusively reserved for aesthetic surgery, dermatology and physiotherapy – is now conquering the cosmetic market because of the easy treatment performance and its efficiency.

During the treatment, under the influence of an electromagnetic field – occurs the phenomenon of polarization causing movement and vibration of ions, atoms, and molecules. The friction of molecules generates endogenous heat (produced inside) and warms up the tissues selectively to a temperature of 45–55°C. Under the influence of thermal energy, the collagen fibers first contract and then tighten. As a result, the regenerating functions are triggered, the fibroblasts are prompted to act and the synthesis of collagen and elastin is stimulated. The skin becomes firm again, the tension and face oval improved. Wrinkles, furrows, and scars get flattened. Under the thermal influence increases also the permeability of

cell membranes and as a result, the absorption of active ingredients contained in the preparations used by the treatment (radio phoresis).

The RF therapy eliminates the signs of aging but also operates as a prevention measure affecting positively both the skin's density and resistance as well.

The radio wave is absorbed by the body in the following sequence:

- fatty tissue,
- skin,
- muscles.

Because of the fact that the fatty tissue gets stronger overheated that the muscles, the treatments involving the RF are a perfect measure for body shaping and slimming. Under the influence of the generated thermal energy – occurs the phenomenon of thermo lipolysis basing on a selective reduction of fatty tissue by shrinking the cell volume. The very fact of improving the blood circulation by the operation of the radio waves is, certainly, not without significance. As a result, the skin is better oxygenated and optimally supplied with nutrients. With the growth of body temperature by 1°C the metabolism is being accelerated by 3,6%, thus, the organism detoxification and the lymphatic system operation are optimized. The above processes boost the cellulite reduction and skin structure regeneration.

The rise of endogenous temperature stimulates the production of endorphin (the hormone of happiness) and causes the muscles to relax, which affects our wellbeing. The **Ice RF Gel** applied during the treatment generates an insulation layer, secures the skin cooling, enables an easy massage, makes the treatment comfortable and safe.

INDICATIONS FOR TREATMENT

- + improvement of tension and condition of the skin of the face
- + lifting of face and neck
- + flattening of wrinkles
- + oval face improvement
- + elimination of cellulite
- + firming of the skin on shoulders, stomach, buttocks and thighs (e.g. after a sudden loss of weight)
- + body-contouring after childbirth
- + burning of adipose tissue

CONTRAINDICATIONS FOR TREATMENT

- × pregnancy, breastfeeding
- × menstruation (treatments on body)
- × cancer (also within 5 years after treatment)
- × epilepsy, multiple sclerosis
- × immunological diseases
- × presence of metallic implants in the treatment area
- × artificial pacemaker, replacement heart valve
- × circulatory insufficiency, hypertension
- × dysesthesia
- × thrombophlebitis, rosacea
- × viral and bacterial infections of the skin
- × tendency to form keloids
- × tendency to digestive tract bleeding
- × overactive thyroid
- × swellings of unknown etiology
- × acute sinusitis
- × IUD–intrauterine device (in case of treatments on stomach)
- × spun gold
- × coagulation disorder
- × implants and fillings in the treatment area

TECHNICAL PARAMETERS

- Range of operation time: 1–30 min
- Frequency: 1 MHz
- Maximum power: 100 VA

DEVICE PANEL

Radio Frequency setting

1. Pulsation rate (-)
2. Pulsation rate (+)
3. Treatment time adjustment (-)
4. Treatment time adjustment (+)
5. **Back** – back to the device selection menu
6. Power control (-)
7. Power control (+)
8. **Start/Stop** – operation mode switch
9. Operating head selection

SETTINGS OF TREATMENT PARAMETERS

1. Select option *Radiofrequency* on the main panel.
2. The display will show the information: **Select program**.
3. Apply **Ice RF Gel** onto the treatment area of the Client's body, then put the operating head of the device into the place to be treated.
4. The head of the device should be all the time in motion. Do not take it away from Client's skin, do not keep it in one place.
5. The pre-set parameters can be changed before the treatment starts.
6. Press **Start/Stop** (8) to confirm the settings.
7. After having finished the above steps, the treatment starts.
8. The treatment clock maintains time until the end of the treatment.
9. As soon as the treatment starts set the pulse intensity at the value clearly noticeable by the Client.
10. The end of treatment is signalled by a short acoustic signal.

MAINTENANCE AND DISINFECTION

After treatment, all the heads of the device are to be carefully wiped dry and disinfected with instant disinfection wipes Hospisept-Tuch. Do not immerse!

The touch-panel should be cleaned with foam for LCD surfaces. The other part of operating panel disinfect with the active foam Anios D.D.S.H. Never use alcohol based products for this purpose.

MESOTHERAPY

PRINCIPLE OF DEVICE OPERATION

Mesotherapy (gr. *mesos* – in the middle, and *therapeia* – therapy) is a treatment based on the introduction of small doses of substances into the pathologically changed area. There is a difference between a needle and needle-free mesotherapy. The forerunner of mesotherapy in the seventies was the French doctor, M. Pistor, who started to apply mesotherapy in aesthetic dermatology. Needleless mesotherapy employs a combination of electroporation related therapy and the introduction of cosmetic preparations (so called treatment cocktails). This method is based on the simultaneous application of the therapeutic effects of ultrasound and impulsive electrotherapeutic stimulus.

In the area of skin undergoing mesotherapy ultrasound head treatment there are:

- local temperature changes,
- changes in intracellular pressure,
- penetration of treatment substances into the subcutaneous tissues.

Electroporation – this is a process occurring in the cell membrane under the influence of high tension electric fields. Under the influence of short duration (lasting for milliseconds) electric field operation there are reversible changes (temporal destabilization) in cell membrane structure (especially in its lipid components), where the pores are formed. This is a safe process causing no damage to cell membranes, allowing the introduction of particles of cosmetic preparations into individual cells. The result is the stimulation of pinocytosis. This is a form of cell absorption of dissolved substances into its interior

(endocytosis). For example, during pinocytosis protein particles or other water-soluble high-molecular substances are taken in. Small liquid droplets are trapped in the microvilli of the cell membrane, from which the fine bubbles rip off on the cytoplasm side. The liquid bubble content permeates slowly into the cytoplasm while the bubbles themselves gradually decrease in size and disappear. Thanks to this the particles of cosmetic preparations function even after the completion of the treatment, prolonging treatment effect. The action time will depend on the activity level of the preparations; the average estimation is at 24–48 h. Electroporation in combination with ultrasound influence allows cosmetic products to permeate into deep layers of the dermis. The results are visible after the first application. This treatment is an alternative to classical needle mesotherapy since it allows to eliminate the side effects connected with injections. This gives the possibility to use a wide range of local face and body treatments. The active ingredients introduced during the treatment must be decomposed through the natural chemical process into components and free radicals, to join the blood circulating radicals at a later stage. The particles can permeate up to 1–2 mm depth.

INDICATIONS FOR TREATMENT

- + wrinkle prevention
- + signs of aging prevention
- + cellulite elimination
- + smoothing of stretch marks and scars
- + skin firming
- + body shaping
- + blood flow stimulation
- + regeneration of irritated couperose skin
- + reduction of skin reddenings

CONTRAINDICATIONS FOR TREATMENT

- × cancer
- × pregnancy, breastfeeding
- × skin inflammations
- × mechanical skin injuries
- × circulation and cardiovascular insufficiency
- × fever, debility and emaciation
- × osteoporosis
- × herpes
- × epilepsy
- × allergy to the ingredients of used preparations
- × presence of foreign metallic matter and electronic implants in the treatment area (artificial pacemaker, dental braces, endoprosthesis, etc.)

TECHNICAL PARAMETERS

- Operation time: 1–40 minutes
- Ultrasound frequency: 1 MHz
- Maximum ultrasound intensity: <3 W/cm²
- Range of pulsation scale (impulse duration modulation): 10%–100% (every 10%)
- Scope of intensity: 0,2 mA–6,2 mA +/- 50% (with a leap of 0,2 mA)
- Impulse duration: from 30 to 900 ms +/- 10%

DEVICE PANEL

Mesotherapy setting

1. Power control (-)
2. Power control (+)
3. Treatment time adjustment (-)
4. Treatment time adjustment (+)
5. **Back** – back to the programme selection; double press – back to the device selection menu
6. Current intensity regulation (-)
7. Current intensity regulation (+)
8. **Start/Stop** – operation mode switch

SETTINGS OF TREATMENT PARAMETERS

- Select on the panel **Mesotherapy**.
- The display will show the information: **Select program:**
Select program Eye/Face/Body.
- Apply the proper active head to the Client body. The head should always be in motion, it must not be distracted from the Client skin nor hold in one place.
- Press the **Start/Stop** button (8).
- The display will show the information: **Set the optimal current: 0,2 mA.**
Start/Stop (8) to approve.
- After completing the above settings the mesotherapy treatment starts.
- The treatment clock measures the time to the end of the treatment. The treatment time was divided into 4 equal treatment stages.
- The end of every stage is signalled by a short acoustic signal.
In the **first step** of the treatment, the current level is always 0,0 mA. During that stage do not change the settings. In the **second stage**, the current automatically change the value, which was chosen on the first settings. Those settings will not change until the end of the treatment. During the **2,3,4 stage**, there is the possibility to change the pulse intensity.
- The end of the treatment is signalled by a short acoustic signal.

MAINTENANCE AND DISINFECTION

After treatment, the heads are to be carefully disinfected with instant disinfection wipes, e.g. with Hospisept-Tuch tissues. Do not immerse!

The touch-panel should be cleaned with foam designed to cleaning LCD screens. The remaining parts of the device should be cleaned with active foam Anios D.D.S.H. Never use alcohol based products for this purpose. All operating heads and the passive electrode wipe dry and disinfect with Hospisept-Tuch. Do not immerse!

TIPS FOR SAFE DEVICE OPERATION AND CORRECT EXPLOITATION

TRANSPORT

The unit can be transported by any available covered means of transportation. During transport, the unit should be protected from moisture. During the transportation do not stack more than 3 units on each other.

The device should be transported in the following conditions:

- **temperature:** from -10°C to 60°C
- **relative humidity:** from 20% to 80%

In the case of a large temperature difference between the temperature of the room in which the device is placed, and transportation temperature, **the unit should be left to the time at which it reaches the temperature of the room in which it is located.** Only after this time, the user can start the device.

PROBLEMS WITH DEVICE OPERATION

PROBLEM	REASON	PROCEDURE
The device cannot be turned on, the display is off	The power cable is not connected to the device	Check if the power cable is firmly connected to the device (if is not loose). Check if it is plugged into the power supply. Check if the cable has no visible damages.
	The fuse is blown	Check if the fuse is not blown (the fuses are situated next to the switch in the back). Attention! Replacing the fuse is only allowed after disconnecting the device from the power supply. The fuse casing can be unscrewed with a screwdriver and removed from the device. If the wire inside the fuse is broken or the fuse inside is dirty, replace with a new one (spare fuses added) and check if the device can be turned on.

CAVITATION PEELING		
The cavitation spatula does not spray or sprays very weak	The power cable of the Skin Scrubber spatula is not connected to the device	Check if the power cable of the spatula is firmly connected to the device (if the cable is not loose and the nut is well threaded on the thread). Attention! Do not connect the spatula into a different device.
	The chosen program is not correct	Make sure that the selected program is "Skin Scrubber" and that the spatula is active (the spatula can also be turned on/off by pressing the button located on the side).
	The spatula's operating power is set too low	Increase the operating power of the spatula on the control panel.
	The treated skin area is not wet	There is not enough tonic on the treated skin area, during the peeling the skin is supposed to be wet.
ULTRA SONIC		
Treatment heads are not working or are working to slow	The power cable of the treatment head is not connected to the device	Check if the power cable is firmly connected to the device (if the cable is not loose and the nut is well threaded on the thread).
	The connected treatment head is not correct	Check if the selected operating program on the panel is correct and if the treatment head is connected to the right input.
	The operating power is set too low	Increase the operating power on the panel.
MICRODERMABRASION		
There is no pressure or the pressure is at the minimum value	Suction tube is not correctly plugged into the filter or nozzle	Check if the suction tube is firmly connected to the nozzle or filter located on the back of the device.
	Negative pressure regulator is at the minimum position	Slide the pressure regulator in the maximum value direction.
	Suction tube is damaged	Cut off the damaged end of the suction tube or insert a new one.
	The filter is not correctly assembled	Check the position of the filter, install it according to the arrows, press.
The device sucks in straight after turning it on	Blocked suction tube or nozzle	Check if the tube is not bended, clean it or replace with a new one, check if the nozzle is not blocked, if necessary clean it.
	The filter is not correctly assembled	Check the position of the filter, install it according to the arrows.
	The filter is overfilled	Replace the filter.
RADIOFREQUENCY		
Treatment heads are not working or are working to slow	The power cable of the treatment head is not connected to the device	Check if the power cable is firmly connected to the device.
	The connected treatment head is not correct	Check if the selected operating program on the panel is correct and if there is the correct treatment head connected.
	The operating power is set too low	Increase the operating power on the control panel.
MESOTHERAPY		
Treatment heads are not working or are working to slow	The power cable of the treatment head is not connected to the device	Check if the power cable is firmly connected to the device (if the cable is not loose and the nut is well threaded on the thread).
	The connected treatment head is not correct	Check if the selected program on the panel is correct and if the head is connected to the right input, check the connection of the flat electrode.
	The operating power is set too low	Increase operating power on the control panel.
Lack of current flow sensation	If the power cable of the flat electrode is not connected to the device	Check if the power cable is firmly connected to the device on the back of it, check the connection of the flat electrode, if necessary replace the connection cable Attention! Impulses can be felt during changing of the treatment settings and for the second stage of the mesotherapy.

WARNING! If despite taking the aforementioned actions the device does not work properly:

- disconnect the device from the power source and contact the maintenance service of the producer,
- do not repair the device on your own, do not check or replace the power cable on your own.

To the service bring the device, the entire set of treatment heads and the warranty card.

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